

TLEGS® SIZING CHART

ANTI-EMBOLISM STOCKING





Measure ankle circumference in centimetres (cm)

Two-fingers width above ankle bone





Check ankle measurement chart for size



# Observe limb proportions


Is the calf in proportion to the ankle?








Are you fitting below knee or thigh length?

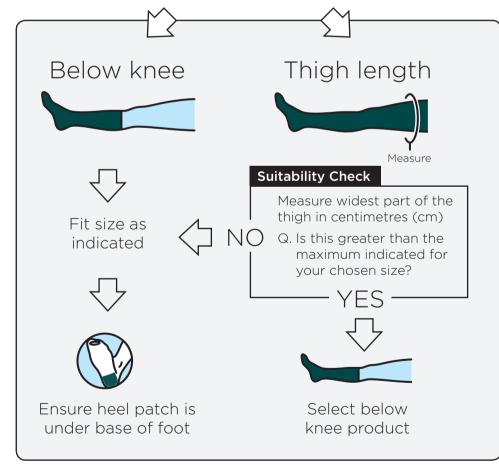

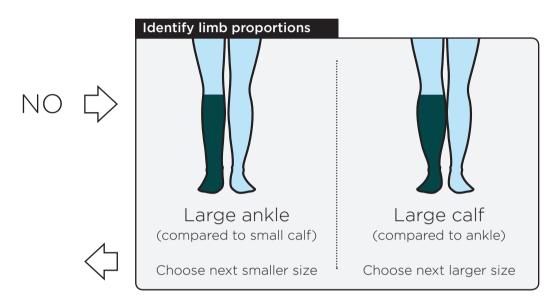

## Refer to packaging for application instructions and contra-indications

## FITLEGS\* AES

Suitab	oility	Chec

Ankle measurement	Size	BELOV NHS Code	V KNEE Order Code	TH NHS Code	IGH Order Code	Maximum Thigh Circumference
16 - 19cm	Slimline	EGD9095	403002			
19 - 23cm	Small	EGD8322	401000	EGD8326	401004	65cm
23 - 26cm	Medium	EGD8323	401001	EGD8327	401005	70cm
26 - 29cm	Large	EGD8324	401002	EGD8328	401006	78cm
29 - 33cm	Ex-Large	EGD8325	401003	EGD8329	401007	82cm
33 - 35cm	Bariatric	EGD8338	403000			

### FITLEGS\* AES GRIP

#### Suitability Check

Ankle measurement	Size	BELOV NHS Code	V KNEE Order Code	TH NHS Code	IGH Order Code	Maximum Thigh Circumference
16 - 19cm	Slimline	EGD9096	403003			
19 - 23cm	Small	EGD8330	402000	EGD8334	402004	65cm
23 - 26cm	Medium	EGD8331	402001	EGD8335	402005	70cm
26 - 29cm	Large	EGD8332	402002	EGD8336	402006	78cm
29 - 33cm	Ex-Large	EGD8333	402003	EGD8337	402007	82cm
33 - 35cm	Bariatric	EGD8339	403001			

Tape measure - NHS code: VWR002 Order code: 405001





Record chosen size and document in notes Re-check measurements and fitting regularly

Griffiths and Nielsen Ltd.

Maydwell Avenue, off Stane Street,

Slinfold, Horsham, West Sussex RH13 0GN



Tel: +44 (0) 1403 799 190 Fax: +44 (0) 845 263 8907 Email: medsales@gandn.com