

Incidin™ OxyFoam S



Broad spectrum, ready-to-use cleaning and disinfection liquid

Broad spectrum cleaner and disinfectant

The advanced formulation of Incidin™ OxyFoam S provides the benefits of a patented* hydrogen peroxide solution with broad spectrum efficacy.

It is effective against a wide range of pathogens and leaves no active residues due to its rapid evaporation rate.

- Active ingredient decomposing into water and oxygen
- Sporicidal & virucidal offering
- Excellent material compatibility

Incidin OxyFoam S is EU-wide approved under BPR (Biocide Product Regulation 528/2012)

- Passing extensive toxicological, ecotoxicological and exposure assessment criteria of BPR
- Fulfilling highest safety standards, ensuring a high level of protection to human health and the environment
- Delivering sustained compliance



*European Patent EP1987121

ECOLAB®
PROTECTING WHAT'S VITAL™



Incidin™ OxyFoam S

Broad spectrum cleaner and disinfectant

Effective

- Sporicidal and virucidal disinfectant

Safe

- Active ingredient decomposes into water and oxygen
- Excellent material compatibility
- Passed extensive BPR assessment

Convenient

- Cleans & disinfects
- Ready to use formats

Microbiological properties

Claim/Conditions	Medical Device	Biocide (PT2)
Bactericidal according to EN 13727 (1 min) / EN 16615 (1 min*)	1 min*	1 min*
Yeasticidal according to EN 13624 (1 min) / EN 16615 (1 min*)	1 min*	
Fungicidal according to EN 13624 (5 min) / EN 16615 (method) (5 min*)	5 min*	5 min*
Tuberculocidal according to EN 14348 (5 min) / EN 16615 (method) (5 min*)	5 min*	5 min* ¹
Mycobactericidal according to EN 14348 (5 min) / EN 16615 (method) (5 min*)	5 min*	
Effective against enveloped viruses according to EN 14476	30 sec*	-- ²
Limited spectrum virucidal according to EN 14476 (Adenovirus - 30 sec, Norovirus - 1 min** / 15 min*)	1 min** / 15 min*	-- ²
Virucidal according to EN 14476	10 min** / 30 min*	30 min* ²
Sporicidal according to EN 17126	60 min**	60 min**

* dirty conditions ** clean conditions

¹ Mycobactericidal efficacy includes tuberculocidal efficacy

² Only full virucidal efficacy was applicable under EN 14476 at the time of dossier compilation and submission.

Indications

Medical Device: Broad spectrum, ready-to-use surface cleaning and disinfection liquid for invasive and non-invasive medical devices.

Biocide: PT02 - Disinfection of non-food contact surfaces in health care applications

- by spraying using trigger sprayer and dry wipe
- by wiping using clean single use cloth/wipe and bucket and by mopping using mop and bucket.

Instructions for use

Medical Device:

750ml: Spray surface from approx. 30cm, ensuring the surface is completely wet. Wipe the surface with a clean wipe and leave to dry. Alternatively, apply the product to a clean wipe, wipe the surface and leave to dry.

5L: Apply the product ensuring the surface is completely wet. Wipe the surface with a clean wipe (e.g. Incidin Dry Wipes System) and leave to dry. Alternatively, apply the product to a clean wipe, wipe the surface and leave to dry.

Applicable to all pack sizes:

Do not rinse. Respect the indicated contact time and conditions for the required antimicrobial activity. If used under clean conditions, clean surface upfront (use clean, fresh cloth/mop for each step). The use of a cleaning and disinfection product does not replace regular cleaning. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass. For reprocessing of medical devices always observe manufacturers' instructions for use, incl. material compatibility. Terminal disinfection of semicritical medical devices (i.e. probes & TEE probes): If immersion is not possible, clean the device and then disinfect by wiping as described above. Rinse thoroughly with water of drinking quality in this case after contact time elapsed.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State.

Biocide:

Always read the label or leaflet before use and follow all the instructions. Product should be applied to a dry surface.

Wet surfaces completely. Do not rinse after use. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass. Product is intended for cleaning and disinfection in one step. When used under clean conditions: clean surface upfront.

Spraying using trigger sprayer and dry wipe:

For optimum results hold bottle upright and spray from a distance of 30 cm. Spray the product onto a dry wipe and wipe small surfaces such as worktops and equipment or spray product onto the surface, then wipe surface with a clean, dry wipe and leave to dry. Always close the nozzle after use. Used wipes must be disposed in a closed container.

Wiping using cloth/wipe or floor mop and bucket:

Pour product in a clean bucket and distribute across surface using single cloth/wipe or across floor using floor mop, let air dry. Used wipes must be disposed in a closed container.

Caution: The product is classified as Biocide as well as Medical Device. Users must respect contact times and conditions for the intended use.

Composition

100g contain the following active ingredient:
1.5g Hydrogen peroxide (CAS N° 7722-84-1).

Storage

Product should be stored in the original container between 0°C and 25°C in a cool, dry location out of direct sunlight.

Period after opening: until end of product shelf life.



FOR PROFESSIONAL USE ONLY.

Medical device and biocides use safely

Please read label and product information.

Incidin™ OxyFoam S

Packaging and ordering information

Product	Packaging unit	Order code
Incidin™ OxyFoam S	6 x 750ml	3115810
Incidin™ OxyFoam S	2 x 5L	3115940
Drain cock (DIN 51) for 5L Canister	1	10033873



The Biocidal Products Regulation (BPR), Regulation (EU) 528/2012:

The BPR replaced the Biocidal Product Directive (BPD) on 1st September 2013 as the legislation responsible for placing biocidal active substances and products on the European Market.

As a regulation, it became legally binding for countries that fall within the scope of EU regulations - all European Union countries as well as Iceland, Liechtenstein, Norway and Switzerland (the European Free Trade Association countries).

The BPR offers a harmonised, centralised authorisation option for biocidal products at a European level versus the BPD. The BPR is designed to control the use of, and the placing on the market of biocidal products.

It involves an assessment of a biocidal product's efficacy, toxicity, and the risk during use. Additionally, it requires robust data to prove the effectiveness and safety of the biocidal product, as well as a list of approved biocidal active substances and suppliers.

EU Biocide Registration Number

EU-0024303-0006 1-5

ECOLAB HEALTHCARE

Winnington Avenue,
Northwich,
Cheshire CW8 4DX
United Kingdom

www.ecolab.com



ECOLAB DEUTSCHLAND GmbH

Ecolab-Allee 1
40789 Monheim am Rhein
Germany

0297

ECOLAB[®]
PROTECTING WHAT'S VITAL™