

# SHARPSGUARD® ECO

24L

## Assembly Instructions



- 1** Match the lip of the lid and the lip of the base for correct assembly.



- 2** Place hands either side of the lid and press until a “click” is heard.  
Turn container 90 degrees and repeat, then press around the rim to ensure the lid is correctly attached to the base.



- 3** Put the temporary closure in place between use, press on mark for temporary closure.



- 4** Lift or flick the central tab to open



- 5** To permanently close Press at back of door and slide thumbs forward following the arrows applying pressure to locking points.



- 6** Do not forget to fill in the label following assembly and permanent closure.



If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44 1706 754980 or [info.healthcareuk@mauserpackaging.com](mailto:info.healthcareuk@mauserpackaging.com) and visit our website [www.daniels.co.uk](http://www.daniels.co.uk).



[www.mauserpackaging.com](http://www.mauserpackaging.com)



[info@mauserpackaging.com](mailto:info@mauserpackaging.com)

REDEFINING SUSTAINABILITY

Unless otherwise stated, the information contained herein is at the specific request of the user and has been made available by Daniels Healthcare.

V01.2023