

# Softdrape\*

Latex *free*  
zone

## Universal Aseptic Dressing Pack

### Description:

Softdrape universal aseptic dressing pack provides you with a one-pack solution to effective wound management. Cost-effective and strategically designed, the pack supports non-touch technique thus minimizing the risk of infection. Softdrape is supplied sterile in easy to open packaging.

### Indications:

- For use in aseptic procedures performed in hospitals and community.

### Features:

- Latex-Free.
- Easy to use.
- Conforms to Infection Control Guidelines.
- Supported by Training and Education.

### Instructions for Use:

1. Open pack onto the clinically clean work surface.
2. Handwash with soap and water.
3. Carefully open outer wrap to create the sterile field.
4. Put apron on.
5. Put hand into the bag to arrange contents.
6. Keep hand in the bag to remove the old dressing.
7. Invert and secure waste bag.
8. Handwash with soap and water.
9. Apply sterile gloves from the pack.
10. Cleanse and dress wound according to local policy.
11. Fold up sterile field and place in waste bag.
12. Seal waste bag and dispose of accordingly.
13. Handwash with soap and water.

### Disposal:

Disposal as per trust policy or local protocol.



 richardson  
healthcare  
value through innovation

## Universal Aseptic Dressing Pack

### Precautions: (including but not limited to)

- Do not use if pack or seal is broken.

### Classification:

Class I Sterile.

### Storage:

Store in a cool dry place not exceeding 25°C.

### Sizes available:

RHC Code	Sizes	NHS Code	UOI	Case Qty
908810	Small	EJA045	20	80
908820	Medium	EJA046	20	80
908830	Large	EJA047	20	80

### Symbols used and their meaning:

Medical device	Manufacturer	Use by date	Batch code
Unique Device Identifier	Date of manufacture	Do not resterilize	Do not reuse/Single use
Keep away from sunlight	Keep dry	Upper limit of temperature 25°C	
Do not use if package is damaged	Not made with natural rubber latex	Recycling	
Single sterile barrier system with protective packaging inside		Sterilised using ethylene oxide	
Authorised representative in the European Union		Attention: Read all warnings and precautions in instruction for use	
CE mark with Notified Body Number 1639		UKCA mark with Approved Body Number 0120	

For further information contact:

☎ 0800 170 1126

✉ [info@richardsonhealthcare.com](mailto:info@richardsonhealthcare.com)

Richardson Healthcare Ltd  
Centennial Park, Elstree, WD6 3TJ, UK



Scan QR code to view product website

