

Tubifast® 2-Way Stretch®

A high quality
tubular bandage for
patients' complete
freedom of movement

Contours to any shaped
limb or torso thanks to
2-Way Stretch® Technology

Washable and re-usable

Self-retaining tubular construction:
no pins or tapes are needed

Easy to use: can be cut to
required length

Latex free


Light elasticity in both vertical
and horizontal directions
allowing patients of complete
freedom of movement¹

Available in 5 colour-coded
widths to fit any part of the body


Tubifast® 2-Way Stretch® tubular bandages are designed for dressing retention patch wrapping (wet or dry) and as an undercast stockinette. Tubifast 2-Way Stretch can also be used for skin covering for any part of the body and in the management of Epidermolysis Bullosa².

- Allows complete freedom of movement
- Self-retaining with no pins or tapes necessary
- Reusable after washing


Instructions for use Tubifast® 2-Way Stretch®

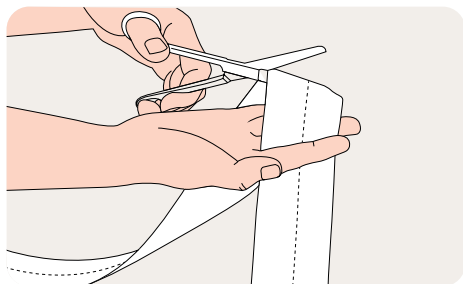

1. Choose the correct bandage width, using the quick reference colour coding, and cut to length.

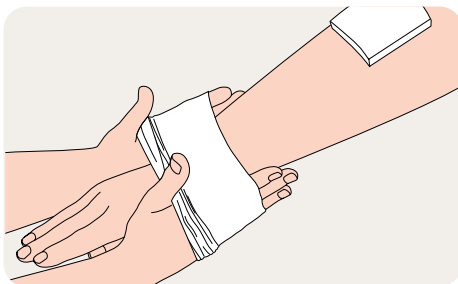

2. Stretch Tubifast® 2-Way Stretch® over the affected area.

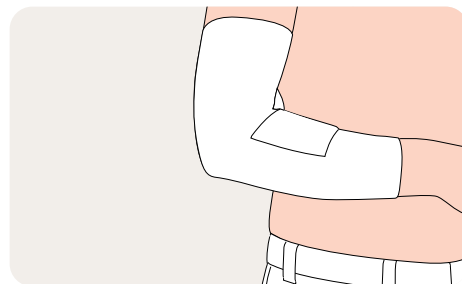

3. Position Tubifast 2-Way Stretch to cover the dressing. Illustrated, step-by-step instructions are available for more advanced applications.

How Tubifast® 2-Way Stretch® works

Tubifast 2-Way Stretch is a self-retaining tubular construction that allows air to circulate over the skin. Tubifast 2-Way Stretch provides a light elasticity in both vertical and horizontal directions allowing patients of complete freedom of movement.

Benefits of Tubifast 2-Way Stretch

- Contours to any shaped limb or torso thanks to 2-Way Stretch® technology
- Allows complete freedom of movement
- Easy to use: can be cut to required length
- Self-retaining with no pins or tapes necessary
- Latex free, non-sensitising and non-irritating
- Reusable after washing
- Available in 5 colour-coded widths to fit any part of the body

Areas of use

Tubifast 2-Way Stretch tubular bandages are designed for dressing retention without the need for pins or tapes; it is particularly suitable for holding dressings in place on difficult areas such as stumps of amputee patients.

The product is also designed for patch wrapping (wet or dry) and as an undercast stockinette. Tubifast 2-Way Stretch can also be used for skin covering for any part of the body and in the management of Epidermolysis Bullosa².

Washing instructions



Fibre content

Viscose, Elastane and Polyamide.



Tubifast 2-Way Stretch ordering information

PRODUCT	SKU	LIMB CIRCUMFERENCE	WIDTH	LENGTH	UOM	NHSSC / NPC	PIP Code
Red Line Small limbs	2480	9-18cm	3.5cm	1m	1	EGP 199	005-0088
	2434			10m	1	EGP 115	
Green Line Small & medium limbs	2481	14-24cm	5cm	1m	12	EGP 205	005-0039
	992485			3m	6	EGP 85000	255-1885
	3555			5m	6	EGP 189	255-5803
	2436			10m	1	EGP 117	
Blue Line Large Limb	2482	24-40cm	7.5cm	1m	12	EGP 200	004-9924
	992486			3m	6	EGP 201	255-1877
	3556			5m	6	EGP 186	255-5811
	2438			10m	1	EGP 119	
Yellow Line Extra Large Limb, Heads, Children's Trunks	2483	35-64cm	10.75cm	1m	12	EGP 188	023-6539
	992487			3m	6	EGP 206	255-1869
	3557			5m	6	EGP 197	255-5829
	2440			10m	1	EGP 121	
Purple Line Adult Trunks	2479	64-130cm	30cm	1m	12	EGP 184	324-1452
	3558			5m	6	EGP 194	324-1486
	2444			1m	12	EGP 017	

References: 1. Eytier C, Gazeau E, Beneteau G, Verfaillie G. Convenience and tolerance of the combination of a soft silicone foam dressing and a 2-way stretch tubular bandage in the management of local wounds. Journal des plaies et cicatrisations 2013;18(88):38-44.
2. Mölnlycke Health Care. Data on file.

Find out more at molnlycke.co.uk

Mölnlycke Health Care Limited: Unity House, Medlock Street, Oldham, Lancashire, OL1 3HS, United Kingdom. Phone: +44 (0) 161 621 3900. The Mölnlycke, Mepilex and Safetac trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2026 Mölnlycke Health Care Limited. All rights reserved. UKWC1401

